





Sizing Balloon

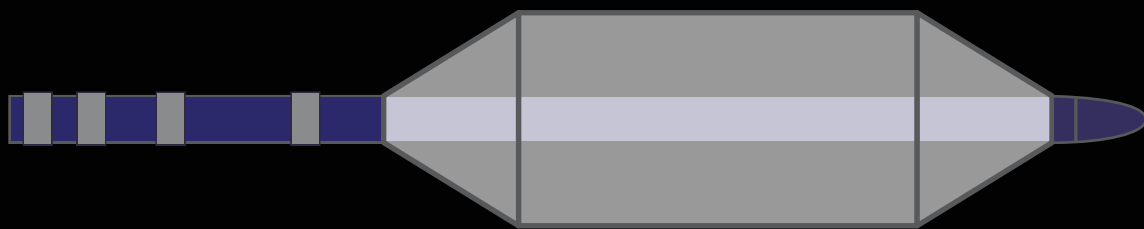



Sizing Balloon

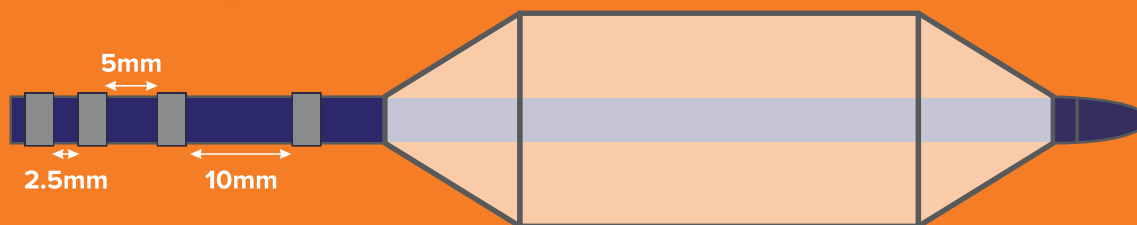

- The Cocoon sizing balloon is specifically designed for sizing atrial communications
- It is a double lumen balloon catheter with radio-opaque marker bands to allow for radiographic measurement. Width of marker band is 1 mm
- The maximum inflation volume of the balloon should not exceed the values shown in the table below

Technical specifications

Catalog number	Max inflated size (mm)	Max inflation volume (cc)	Balloon length (mm)	Shaft diameter (inch)	Shaft length (cm)
CAB24	32	36	55	0.090	70
CAB34	44	88	75	0.090	70

CAB24 recommended for pediatrics

CAB34 recommended for adults

Sizing balloons chart

24 mm		34 mm	
Inflated diameter (mm)	Inflation volume (cc)	Inflated diameter (mm)	Inflation volume (cc)
24	20	34	50
25	22	35	52
26	24	36	54
27	26	37	57
28	28	38	61
29	30	39	65
30	32	40	68
31	34	41	73
32	36	42	78
		43	83
		44	88
Maximum inflated size /inflation volume			

Note

Do not inflate the Cocoon sizing balloon beyond the "stop-flow" point or beyond the maximum inflation volume. A waist in the balloon could appear without the cessation of flow. This would occur if there is more than one ASD. Sizing should be done based on "stop-flow" and not just on the appearance of a waist

Caution: This product is intended for use by or under the direction of a physician. Prior to use, refer to the "Instructions for use" supplied with these devices for indications, contraindications, side effects, suggested procedure warnings and precautions. As part of our continuous product development policy, we reserve the right to change product specifications without prior notification. Information contained herein is for distribution outside the U.S. & Japan only.

Check the regulatory status of the device before distribution in areas where CE marking is not the regulation in force. Tests performed by and data on file at Sahajanand Medical Technologies Limited. Illustrations are artist's representations only and should not be considered as engineering drawings or photographs. Photos on file at Sahajanand Medical Technologies Limited.

Cocoon product range is manufactured by Vascular Innovations Co., Ltd, Thailand, a SMT group company. Cocoon is a trademark of Vascular Innovations Co., Ltd. Cocoon sizing balloon is currently not approved by USFDA and is not available for sale in USA.

Disclaimer: © 2024 Sahajanand Medical Technologies Limited – All rights reserved. Specifications are subject to modification, revision and improvement.

This brochure is applicable only for Indian market.

SMT CardioVascular Private Limited

Plot M79, M80, M88, M89, M90, Medical Device Park, Sultanpur, Patancheru, District: Sangareddy, Hyderabad- 502 319, Telangana (India)

• www.smtpl.com

